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Self Adhesive Dual Cure Luting Cement

DC LUTE-X SE AUTOMIX 4:1 is a self adhesive dual cure luting cement for permanent cementations.

When DC LUTE-X SE AUTOMIX 4:1 is used, conditioning and bonding of the tooth structure are not necessary. DC LUTE-X SE AUTOMIX 4:1 is self neutralizing.

DC LUTE-X SE AUTOMIX 4:1 is based on methacrylate resin and inorganic fillers. The total filler load is 50% wt, the total filler volume is 45 %. The mixing ratio based on volume is 4 parts base and 1 part catalyst.

Indications

- crowns and bridges made of metal and metal ceramic
- crowns and bridges of reinforced ceramics (e.g. Zirconia)
- all-ceramic crowns and bridges produced by using Cerec®
- inlays and onlays of ceramic, metal and composite
- posts
- metal, metal ceramic and reinforced ceramics (e.g. Zirconia) on implant abutments

Contraindications

Thin all-ceramic restorations of feldspathic ceramic or other low fusing ceramics, veneers, Maryland-bridges.

Preparing the MINIMIX-Syringe

Remove the MINIMIX-syringe from the seal-pack and discard the bag. Note the date of removal on the syringe label.

Remove the cap of the MINIMIX-syringe and throw it away (**do not use it again!**). Squeeze out a small quantity of paste on a mixing pad to equal the base and catalyst paste in the MINIMIX-syringe and discard it.

Attach a 4:1 mixing cannula and turn 90° until it locks in position. The Material is now ready for application.

The working time (23°C (74°F)) in the self cure mode is 1:30 minutes from start of mixing.

Note:

Store used cartridge with fixed used mixing cannula in the dark. Discard a peppercorn-size quantity of material. This has to be done for each new mix.

Pulp protection

Pulp capping with DC LUTE-X SE AUTOMIX 4:1 is contraindicated. Cavity floor of deep excavations should be covered with a thin layer of calcium-hydroxide material.

1. Cementation of Inlays, Onlays, Crowns and Bridges

1.1. Preparing

Prepare the luting side areas of the all-ceramic inlays, onlays, crowns and bridges according to manufacturer instructions and primed with a silane priming agent.

The luting side areas of metal, zirconia or composite restorations should be prepared according to manufacturer instructions.

1.2. Cementing of the indirect restoration

1.2.1. Preparation of cavity/tooth stump

For a sufficient retention the height of the prepared stump must be at least 4 mm with a steep preparation angle (maximum 6 degrees).

Prior to final cementation, clean the prepared stump or cavity thoroughly with pumice slurry, rinse with water and slightly dry in 2-3 intervals with oil free air. **Do not overdry!**

1.2.2. Cementing of Inlays, Onlays, Crowns and Bridges

For cementing (preparation of the restoration see 1.1.) apply a uniform coat (0.5 mm) of DC LUTE-X SE AUTOMIX 4:1 on the luting side of the inlays, onlays, crowns and bridges and on the prepared tooth areas. Seat the restoration under slight pressure.

Remove excess material after brief light exposure (approx. 5 sec). Light cure all marginal areas of the restoration from each direction (mesio-oral, disto-oral, mesio-buccal, disto-buccal) for **20 seconds** with a polymerization unit (wavelength range of 400-500 nm and light intensity of at least 1000mW/cm²).

2. Cementation of Posts

2.1. Preparing of the root canal

Use of a rubber dam to isolate the tooth is strongly recommended.

Prepare the selected post according to manufacturer directions.

Treat the root canal endodontically and clean it in the usual manner (e.g. with sodium hypochlorite solution, approx. 3%). Rinse with water and dry with paper points.

2.2. Filling of the root canal

Prepare the MINIMIX-syringe as described above and attach an Endo-Tip to the mixing cannula. Insert the Endo-Tip as deep as possible into the root canal and apply DC LUTE-X SE AUTOMIX 4:1 starting apical. Keep the end of the Endo-Tip in the material to avoid bubbles. When the root canal has been completely filled remove the Endo-Tip from the material.

If necessary apply a thin layer of DC LUTE-X SE AUTOMIX 4:1 on the post. Place the post in the filled root canal and hold in position under moderate pressure. Light cure the coronal part of the cemented post for **20 seconds** with a polymerization unit (wavelength range of 400-500 nm and light intensity of at least 1000mW/cm²).

Additional Notes

- Do not use any resin to adjust viscosity of luting cement.
- Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the material. Neither store the material in proximity of eugenol containing products, nor let the composite allow coming into contact with materials containing eugenol.
- Contact of resin pastes with skin should be avoided, especially by anyone having known resin allergies.
- Catalyst paste: Contact with eyes may cause severe eye damage. Wear eye protection. In case of contact with eyes rinse immediately with plenty of water and seek medical advice.
- Cerec® is not a registered trade mark of S&C Polymer Silicon-und Composite Spezialitäten GmbH

Storage

Store in the original seal-pack at 10- 25 °C. After opening of the seal-pack, use DC LUTE-X SE AUTOMIX 4:1 within 6 months and before the end of expiry date. Avoid constantly high humidity. Do **not** store in the refrigerator.

Do not use after expiry date.

Warranty

First Scientific Dental GmbH warrants this product will be free from defects in material and manufacture. First Scientific Dental GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and First Scientific Dental GmbH's sole obligation shall be repair or replacement of the First Scientific Dental GmbH product.

Limitation of Liability

Except where prohibited by law, First Scientific Dental GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!

For dental use only!