LC CALCIUM HYDROXID LINER

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LC CALCIUM HYDROXID LINER

Light cure radiopaque liner with calcium hydroxide

Instructions for Use (EN)

LC CALCIUM HYDROXID LINER is a light cure, radiopaque liner with calcium hydroxide.

Indications:

- indirect pulp capping
- lining of cavities for following filling procedures

Application

Clean and dry the cavity thoroughly. Avoid contamination

Screw off the cap of the syringe and substitute it by a NeedleTip. After use discard the NeedleTip and attach the cap to the syringe again. NeedleTips are for single use only.

Note:

In direct proximity to the pulp (direct pulp capping) application of an aqueous calcium hydroxide liner (e.g. CALCIUM HYDROXID LINER) is recommended.

To enhance bonding of LC CALCIUM HYDROXID LINER use of a bonding agent (e. g. LC ETCHBOND X1) according to the corresponding instructions is recommended. In areas with a thin dentin layer over the pulp do without bonding (pulp protection).

Apply LC CALCIUM HYDROXID LINER in a thin layer (max. 1 mm) and light cure for **40 seconds** by using a polymerization unit (wavelength range 400-500 nm) with a light intensity of at least 1000mW/cm².

Note: Avoid placing LC CALCIUM HYDROXID LINER on enamel or the margins of the cavity.

After curing apply the filling according to manufacturer instructions.

Storage

Store at 4 - 25 °C (39 - 77 °F) protected from light and moisture. Close syringe immediately after use. Do not use after expiry date.

Additional Notes:

- When using LC CALCIUM HYDROXID LINER wear protective glasses and cover the patient eyes.
- Unpolymerized materials may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin, mucous membrane and eyes.
- In case of contact with eyes, rinse with copious amounts of water and consult a physician immediately.
- Avoid contact with skin. If accidental contact occurs, wash immediately with water.
- Do not use any resin to adjust viscosity of LC CALCIUM HYDROXID LINER.
- Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the material. Neither store the material in proximity of eugenol containing products, nor let the material allow coming into contact with materials containing eugenol.

Warranty

First Scientific Dental GmbH warrants this product will be free from defects in material and manufacture. First Scientific Dental GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and First Scientific Dental GmbH's sole obligation shall be repair or replacement of the First Scientific Dental GmbH product.

Limitation of Liability

Except where prohibited by law, First Scientific Dental GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!

For dental use only!